DE

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EN

Hydrogen peroxide system for cleaning, disinfecting, neutralizing and storing all types of contact lenses, except for colour and/or cosmetic lenses. Removes proteins.

IMPORTANT DIRECTIONS FOR USE AND SAFETY INFORMATION, PLEASE READ CAREFULLY AND KEEP FOR FUTURE REFERENCE. IMPROPER CARE OF CONTACT LENSES CAN LEAD TO SERIOUS EYE DAMAGE.

The 3% hydrogen peroxide solution with buffer system is always supplied together with the contact lens case containing the platinum catalyst (platinum disc).

ALWAYS USE THE SUPPLIED CONTACT LENS CASE CONTAINING THE PLATINUM CATALYTIC DISC. The platinum catalyst is fixed under the lens holder. Always check to make sure that it is in place. It is essential for neutralizing the hydrogen peroxide system. Please never use a different case for your contact lenses.

DIRECTIONS FOR USE

Before handling the contact lenses, wash your hands thoroughly, rinse completely, and dry them with a clean towel.

• Open the compartment of the lens holder marked with the letter "L", and place the left lens on the raised convex mount. When you close the compartment, be careful that the lens does not get caught in it.

• Repeat the above step for the right side and place the right lens in the compartment marked with the letter "R".

• Fill the case with hydrogen peroxide solution up to the filling line. **DO NOT UNDERFILL OR OVERFILL** the case!

• Insert the lens holder into the contact lens case and close the case tightly, without shaking it!

• Keep the case in an upright position.

• You can tell that the cleaning system is working properly if air bubbles form when the solution comes into contact with the catalyst inside the lens case; otherwise, do not use the product.

• The contact lenses must be kept in the solution for at least 6 hours or overnight to ensure that the solution is completely neutralized.

• After the contact lenses have been immersed in the solution for the required time, they are ready to be worn again. For particularly sensitive eyes, we recommend rinsing the lenses with a preservative-free saline solution before use.

• Pour out the used solution from the lens case.

• Rinse the case and the lens holder with saline solution and leave them open to dry. **NEVER USE TAP WATER TO RINSE THE CASE AND THE LENS HOLDER.**

• If the contact lenses remain in the solution for more than 24 hours, the cleaning process must be repeated prior to wearing them.

CONTRAINDICATIONS (reasons not to use the product)

Do not use if you are sensitive or allergic to any of the ingredients in this product.

PRECAUTIONS

• Do not use the product after the expiration date has passed.

• Do not use or combine any other products together with the hydrogen peroxide system.

• Only use the case included in the package and replace it each time you purchase a new package.

• Do not use the lens case with other solutions.

• Make sure that the platinum-coated catalyst disc is inside the lens case.

• Never remove the lid of the lens case during the disinfection process.

• After first opening of the bottle, use the solution in the bottle within 2 months. **DO NOT USE THE SOLUTION AFTER IT HAS BEEN OPEN FOR MORE THAN 2 MONTHS.**

• Close the bottle tightly after each use and store at temperatures between 5 and 25°C (40 and 75°F), away from sunlight and heat sources. The bottle may expand at higher temperatures, if the bottle can be opened with care.

• To avoid contamination, make sure the tip of the dropper does not come into contact with other surfaces.

• **KEEP OUT OF REACH OF CHILDREN.**

• Do not use the solution if the seal around the bottle is not intact (broken, missing or tampered with).

• If the hydrogen peroxide solution spills, rinse the area with plenty of water to prevent stains from forming.

WARNINGS

• Never use on colour contact lenses.

• **NEVER USE TAP WATER TO RINSE THE CONTACT LENSES,** only use a saline solution or a multipurpose solution.

• **NEVER PUT NON NEUTRALIZED HYDROGEN PEROXIDE SOLUTION DIRECTLY INTO YOUR EYES, AS IT CAN CAUSE SEVERE BURNING.** If non neutralized solution gets into your eye, flush the eye thoroughly with plenty of tap water.

• Always use fresh hydrogen peroxide solution. Never reuse the hydrogen solution.

• If you accidentally swallow non neutralized solution, immediately drink large quantities of water and consult a doctor.

• Before taking any medication, consult your eye care professional or ophthalmologist.

SIDE EFFECTS

Should any side affects appear such as inflammation, permanent irritation or other problems, remove the contact lenses and never use the hydrogen peroxide solution again. Consult your eye care professional or ophthalmologist.

INGREDIENTS

Hydrogen peroxide 3%, phosphate buffered solution, stabilized with ethylene diamine tetra acetic acid 0,03% and phosphorous acid, sodium chloride, boric acid, purified water.

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